

3' ROLLUP™ RAMP

STOCK NO. ROLLUP 3

EZ-ACCESS®

INSTRUCTIONS FOR USE


1. Maximum weight limit is **600 pounds**.
2. Read and follow all instructions before use.
3. Do not use if ramp is damaged or unstable.
4. For use with wheelchairs and scooters.
5. Remove all parts from box. Check to make sure you have received the following: ROLLUP™ tread plate; side-rail supports (2 each); storage bag; clevis pins (two each).
6. Unroll tread plate (DIAGRAM 1) by either standing it on end OR lying down and unrolling (similar to the unrolling of a sleeping bag). CAUTION: Be careful not to pinch fingers when unrolling.
7. Stand tread plate on edge (DIAGRAM 2).
8. Identify latches located on back of tread plate; these will be used to attach side rails to tread plate (DIAGRAMS 2 & 4). NOTE: Each latch features color-coded dots that need to be aligned with corresponding dots on side rails during assembly.
9. To attach first side rail (DIAGRAM 3), match colored dot on side rail with corresponding colored dot on latch (DIAGRAM 2) (i.e., green to green, pink to pink, etc.).
10. Support tread plate with leg or knee, making sure latch is facing out and away from body.
11. Take side rail in both hands, making sure that the silver catch on the side rail is facing toward latch so that you can latch side rail to tread plate. Place side rail over edge of tread plate. NOTE: To allow side rail to slip on easily, ensure that tread plate is as straight as possible (DIAGRAMS 2 & 3).
12. Using latch on tread plate and catch on side rail, attach side rail to tread plate (DIAGRAM 4). IMPORTANT: After attaching, be sure latch is securely locked into place. Double check to ensure that the colored dots on side rails and tread plate are aligned and latched together with their corresponding color (i.e., green to green, pink to pink, etc.).
13. Flip ramp to opposite side and attach second rail, repeating steps 7-12.



14. **WARNING:** Once both side rails are locked securely into place, double check that they are correctly attached. This can be done by ensuring that the notched ends of the side rails are at the top of the ramp (labeled TOP) (DIAGRAM 5). The notched ends form a lip to secure ramp when placed on a step or curb (DIAGRAM 5).



15. **WARNING:** Ensure that the top plate is secure before using. We recommend attaching the ramp temporarily to the landing using the provided clevis pins (drill 5/16" hole in landing) or semi-permanently using screws through the pre-drilled holes at the top of each section. (DIAGRAM 2). To prevent ramp from becoming dislodged during use, make sure that the top plate overlaps the resting surface as far as possible. The surface holding the top plate needs to be substantial; make sure that the surface and supporting structure is of sound construction
16. IMPORTANT: USE RAMP ONLY WITH QUALIFIED HELPER.
17. Always use your lap belt.
18. An incline chart is provided on the reverse side of this document. However, it is intended only as a guideline. It is important that you refer to your equipment's (wheelchair or scooter) Owner Guide for the proper degree of incline/decline and chair direction before attempting ramp use. Never exceed its recommendations.
19. STORAGE: Detach both side rails from tread plate. To roll up tread plate, begin rolling at the top flap. Place ramp in provided storage bag.

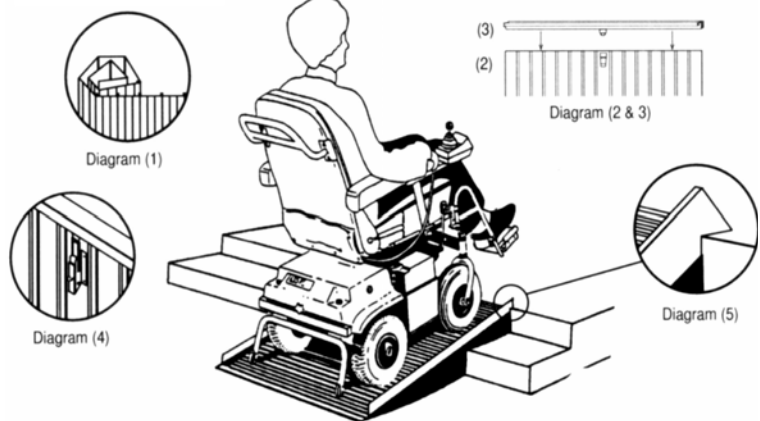

1. EZ-ACCESS, a division of Homecare Products, Inc., herein referred to as Manufacturer, warrants that the equipment is free from defects in material and workmanship under normal use and service. This limited warranty is effective for a period of 3-years from the date of original retail purchase (**original dated purchase receipt** is required) and applies to metal structure only (excludes powder coated finish and applied non-skid tread). The provisions of this warranty shall not apply to any equipment which has been subject to misuse, vandalism, negligence, alteration, accident, or improper installation, or which has been repaired outside Manufacturer's place of business in any way as, in the reasonable judgment of Manufacturer, to adversely affect its performance and reliability, nor to normal deterioration due to wear, tear and exposure, corrosion, damage caused by rain, fire, earthquake or other natural causes or acts of nature.
2. Manufacturer's obligation under this warranty is limited to repairing or replacing, at its option, any equipment which is returned to Manufacturer's place of business with return authorization number (SEE #4 FOR DETAILS) and which, upon examination, shall disclose to Manufacturer reasonable satisfaction to have been thus defective. Manufacturer will make the repair or replacement of defective parts under this warranty without charge for parts or labor.
3. If Manufacturer finds that any equipment has not provided the warranted service life, Manufacturer will, as its sole obligation and the user's sole remedy for breach of this warranty, repair or replace the unit, at its option, for a charge prorated on the following basis. The user will be allowed a credit against the list price of equivalent equipment at the time of return to Manufacturer, in proportion to the percentage of the warranted service life remaining at the time of return to Manufacturer. Purchaser will be responsible to pay the balance of the list price of equivalent equipment; and Manufacturer reserves the right to require payment prior to delivery of the repaired/replacement equipment. Manufacturer's whole warranty responsibilities are summarized below (return **shipping/delivery charges** to be paid by the Purchaser):

YEARS 0-1:	unit repaired or replaced free of charge.
YEARS 1-2:	50% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
YEARS 2-3:	25% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
4. The attached registration card must be mailed within 10-days after purchase to the Manufacturer in order to validate the warranty. If Purchaser discovers a defect, he/she must call Manufacturer at 253-249-1101 and request an RA number to return, at his/her own expense, the merchandise to Manufacturer. Additionally, digital images of merchandise (close up of defect, plus photo of overall product) must be submitted (an email address will be provided at the time an authorization number is given). The duration of any implied warranty of merchantability or fitness for a particular purpose shall be limited in all respects to the duration of the limited warranty and the warranty described above shall be in lieu of any other express warranty. Some states do not allow limitation on how long an implied warranty will last so the above limitation may not apply to you. Manufacturer neither assumes nor authorizes any other person to assume for it any other liability in connection with Manufacturer.
5. The buyer's sole and exclusive remedy against the Manufacturer shall be for the repair or replacement of defective merchandise as provided above. No other remedy, including but not limited to, incidental or consequential damages for lost profits, lost sales, injury to property, or any other incidental or consequential loss shall be available to him/her. Because some states do not allow the exclusion or limitation of incidental or consequential damages, the above limitation or exclusion may not apply to you. Before using, user shall determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.
6. This warranty gives you specific legal rights. You may also have other rights, which vary from state-to-state.

